

FIZE kUO® Pocket Guide

## Placing a new foley and FIZE kUO® system (ICU and OR)

- 1 Connect the device to a power source, power it on, and place as desired (not on the floor)
- 2 To continue with a current patient - Press "Continue with current patient". For a new patient - press "Discard and start new patient" and confirm
- 3 Set up your sterile field and insert Foley per hospital SOP
- 4 Insert cassette into console
- 5 Unclamp and press "Start" on the screen



**New  
Foley**

Transferring  
a patient

Receiving  
a patient

Ambulation  
and procedures

Discontinuing  
Foleys

Customer  
support



FIZE kUO® Pocket Guide

## Transferring your patient to another department

- 1 Best practice is to transfer patient with device
- 2 If not possible, eject cassette and travel, leaving console behind

**NOTE:**

Once out of console, line is subject to gravity and backflow – Keep below bladder.

- 3 Transfer as per your hospital's SOP



New  
Foley


**Transferring  
a patient**

Receiving  
a patient

Ambulation  
and procedures

Discontinuing  
Foleys

Customer  
support

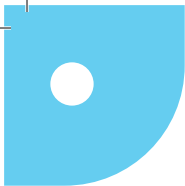

FIZE kUO® Pocket Guide

## Receiving a Patient with FIZE kUO® kit

- 1 Plug into power- then power on, and place as desired (not on the floor)
- 2 Insert cassette into console
- 3 Select “Continue with current patient”
- 4 Ensure clamp is open and press “Start” on the screen
- 5 Patient identifier, weight and last 12 hours of monitoring will automatically transfer to the console



New  
Foley

Transferring  
a patient

**Receiving  
a patient**

Ambulation  
and procedures

Discontinuing  
Foleys

Customer  
support



FIZE kUO® Pocket Guide

## Ambulation and procedures

- 1 Best Practice is to travel with the console  
If not possible\*, eject cassette to leave the console behind
- 2

**NOTE:**

The kit and console must stay connected to operate. Disconnecting them stops monitoring and urine output won't be measured. During this time, the kit drains by gravity.

- 3 Travel as per SOP

\* Consoles are not MRI safe, but kits are



New  
Foley


Transferring  
a patient

Receiving  
a patient

**Ambulation  
and procedures**

Discontinuing  
Foleys

Customer  
support

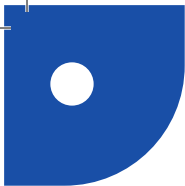

FIZE kUO® Pocket Guide

## Discontinuing Foleys

- 1 Press "Eject" on your console to release the cassette
- 2 Remove Foley as per your hospital's SOP
- 3 Discard the kit and catheter according to hospital SOP. Power off the device and process the used console as per hospital protocols



New  
Foley


Transferring  
a patient

Receiving  
a patient

Ambulation  
and procedures

**Discontinuing  
Foleys**

Customer  
support

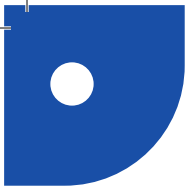

FIZE kUO® Pocket Guide

## Feedback and Support Resources



**We value your feedback!**

Please tell us about your  
experience using the FIZE kUO®

### **Service Number**

**(833) 307 4002**

We're here to support you 24/7

Please feel free to call if you have any questions

LB-0016 Ver1



New  
Foley

Transferring  
a patient

Receiving  
a patient

Ambulation  
and procedures

Discontinuing  
Foleys

**Customer  
support**